

## BUSTER Body Suit for Rabbits

An ideal alternative to an E-collar for rabbits recovering from surgery or minor injuries.

Provides optimal protection and comfort while preventing the patient from reaching tender areas on the body, promoting faster healing and recovery.

### Perfect for

- Postoperative care
- Wound care
- Neuter recovery
- Hot spots and skin diseases

### Enables

- Optimal security
- Freedom of movement
- Stressless handling
- Insertion of pads for fluid-control

### Provides

- Fast recovery
- Less stress
- Mental well-being
- Comfortable fit



Scan QR code to  
watch the video



**Instructions for Use**

1. Insert the head into the body suit
2. Insert each front leg separately into the openings
3. Pull the body suit as far as possible over the body
4. Place the closing flap between the rear legs and insert the tail through the tail opening
5. Attach the closing flap on the touch fasteners on the loin and secure the press studs



**AVAILABLE IN 6 SIZES**

Cat. No	Description	Pk Size	Rabbit Weight Kg	Rabbit Weight Lbs
274048	BUSTER Body Suit for Rabbits, 3XS, black	1/pk	0.6 - 0.8	1.3 - 1.8
274049	BUSTER Body Suit for Rabbits, 2XS, black	1/pk	0.8 - 1.5	1.8 - 3.3
274050	BUSTER Body Suit for Rabbits, XS, black	1/pk	1.5 - 2.0	3.3 - 4.4
274057	BUSTER Body Suit for Rabbits, S, black	1/pk	2.0 - 3.5	4.4 - 7.7
274058	BUSTER Body Suit for Rabbits, M, black	1/pk	3.5 - 4.5	7.7 - 9.9
274059	BUSTER Body Suit for Rabbits, L, black	1/pk	4.5 - 5.5	9.9 - 12.1
274078	BUSTER Body Suit for Rabbits, Starter Pack, 3XS-L, black	6/pk		



Scan QR code to learn more

