GB

Translation of the original instructions

Akkurata GT405 (HS71) / Vega GT410 (HS71)

F

Traduction de la notice originale

Akkurata GT405 (HS71) / Vega GT410 (HS71)

E

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Akkurata GT405 (HS71) / Vega GT410 (HS71)

J

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Akkurata/Vega

TW

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Akkurata GT405 (HS71) / Vega GT410 (HS71)

English	4
Translation of the original instructions	
Français	26
Traduction de la notice originale	
Español	48
Traducción del manual original	
日本語	70
取扱説明書	
國語	92
姿钁璧讨鈕	

Contents

1.	Names of parts.....	7	6.	Changing the rechargeable battery	19
2.	Safety	7	7.	What should I do if	20
2.1	Explanation of warnings	7	8.	Technical data.....	22
2.2	Intended use.....	8	9.	Disposal.....	23
2.3	Important information about dangers	8	10.	United Kingdom	23
2.4	General safety information	9	11.	Two-year guarantee.....	24
2.5	Safety during operation.....	12			
2.6	Lithium ion battery	12			
2.7	Cleaning, care and storage	13			
3.	Preparation.....	14			
3.1	Charging	14			
3.2	Storing the power cord	15			
3.3	Storing the comb attachments.....	15			
4.	Operation	16			
4.1	Function lock	16			
4.2	Mains operation.....	16			
4.3	Switching On and Off	16			
4.4	Setting the cutting length.....	17			
4.5	Comb attachments.....	17			
5.	Cleaning and care	17			

Thank you

for choosing the **Akkurata/Vega**, the new hair trimmer from Aesculap.

It embodies the latest technology. This professional hair trimmer is innovative and design oriented. When developing the hair trimmer, the requirements for a professional trimmer used in a hair salon were taken into consideration.





In addition to optimum cutting performance, the trimmer provides the following advantages:

- Cordless or mains operation
- Fast charge in less than one hour
- 3-stage cutting length adjustment
- Battery charge indicator

We hope you enjoy working with the trimmer.

Best wishes

Aesculap Schermaschinen GmbH

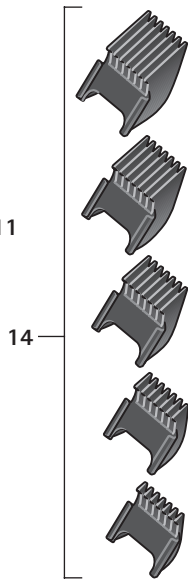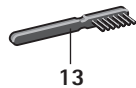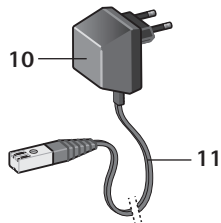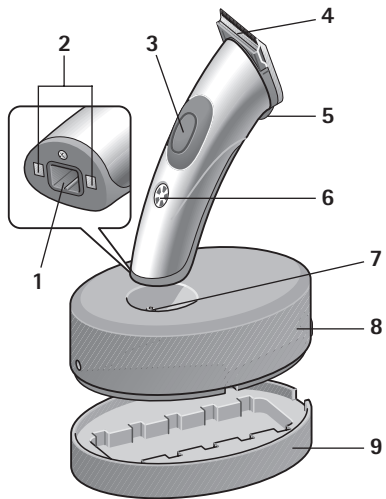1. Names of parts

- 1 Power cord socket on trimmer (for mains operation)
- 2 Charging contacts on trimmer
- 3 On/Off switch
- 4 Blade set
- 5 Setting lever for cutting lengths
- 6 Charging indicator / battery charge indicator
- 7 Charging contacts on charging unit
- 8 Charging unit
- 9 Comb attachment holder
- 10 Mains adapter
- 11 Power cord for charging unit and mains operation
- 12 Oil bottle
- 13 Cleaning brush
- 14 Comb attachments 4, 8, 12, 16, 20 mm

2. Safety

Please read the instructions carefully, especially the chapter on "Safety", **before** using the trimmer. This will protect you from possible dangers and the appliance from damage due to incorrect operation.

Keep these instructions in a safe place for future reference. If you pass on the appliance to someone else, be sure to hand over these instructions as well.

2.1 Explanation of warnings

In these instructions you will find the following warnings:

DANGER

High risk. Failure to observe the warning can cause serious injury.

WARNING

Medium risk. Failure to observe the warning may cause damage to property.

NOTE

Low risk. Points that you should observe when using or handling the appliance.

2.2 Intended use

- The trimmer machine is used for trimming small animal hair.
- The appliance is designed for indoor use only. Do not use outdoors (garden, patio etc.).
- The appliance was developed for professional use in grooming-salons and veterenary practice. Provided the safety information is observed, it can also be used in the home.

2.3 Important information about dangers**Danger of electric shock!**

- The appliance must never be used or charged in the bath or shower, or over a full wash basin or other water container.

- Keep the appliance, charging unit and the power cord away from moisture, water and other liquids, and do not operate with wet hands.

- Do not immerse the appliance, charging unit or the power cord in water or rinse them off under running water.
- If the appliance has fallen into water, it must not be used after that.
- Additional protection is afforded by installing a **residual current device** with a rated tripping current of less than 30 mA in your home electrical system. We recommend you get an electrician to install a residual current protection system to protect the circuits.

WARNING

The shaver head can become hot after extended use!

NOTE

The clippers for animals may only be operated with a power supply unit in accordance with the specifications on the identification plate.

2.4 General safety information

DANGER

- Never use the appliance if:
 - the trimmer, charging unit or the power cord are damaged.
 - the blade set is damaged or breaking up, as this could cause injuries to the skin.
 - the appliance has fallen or been dropped, shows signs of damage or is not functioning properly.
 - the appliance is so badly damaged that electrical parts are exposed.

In all such cases, disconnect from the mains immediately and inform Technical Customer Service.

- Check whether the mains voltage agrees with the voltage specified on the rating plate.
- Unplug the mains adapter if a fault occurs during charging or when running on mains power.

- Only operate device with the original power supply.

If the device has the marking  C20, any C20 model power supply can be used.

Trimmer machines for animals may only be used by trained personnel.

- Keep the appliance out of reach of children.
- Also keep the packaging material, e.g. plastic film, away from children.
- Lay the power cord so that nobody can trip over or tread on it.
- Protect the power cord from damage. Never pull on the power cord. When pulling the mains adapter out of the power socket, always grasp the adapter itself. Keep the cable away from hot surfaces. Check the power cord regularly for damage.
- Operate or charge the device only between 0°C and 40°C.
- To avoid risks, repairs must only be carried out by authorised Aesculap Customer Service Centres.

Small parts which can be swallowed!

- The cover of the oil bottle is a small part that can be swallowed. Keep it away from children younger than 36 months old.

- If the power cord **11** is damaged, the complete cable with mains adapter must be replaced. Use only the original spare part, which you can obtain from Aesculap Customer Service.
- Unplug device from the power supply for maintenance and part replacement.
- Do not open the trimmer, except to change the rechargeable battery (see "Changing the rechargeable battery" on page 19).
- The blade set must not be dismantled.

NOTE

The safety of Aesculap electrical appliances is in line with the recognised state of the art and the German Equipment Safety Act.

Light-emitting devices such as LEDs or infrared light do not exceed Laser class 1 of IEC 825-1.

Given the present state of the art, components of electrical appliances are subject to a certain degree of wear over time. To ensure perfect functioning and safety, we therefore recommend regular inspection by the Technical Customer Service of Aesculap Schermaschinen GmbH.

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Further service addresses you can demand from the mentioned above address.

2.5 Safety during operation

DANGER

- Do not touch the blade set while the appliance is active.

- The appliance must be switched off whenever:
 - you put it down, e.g. to recharge it or stop cutting hair for a short period.
 - you fit or remove a comb attachment.
- The cable must not touch the blade set during operation.
- Use the trimmer only in combination with the original accessories (charging unit, comb attachments, rechargeable battery, power cord with mains adapter etc.).

2.6 Lithium ion battery

The appliance ships with an integrated, rechargeable lithium ion battery, which is characterized by very high capacity combined with low weight.

The battery does not suffer from any "memory effect" and offers a long service life, if handled correctly.

DANGER

Incorrect handling of the lithium ion battery can cause the destruction, explosion or ignition of the battery.

- Therefore, always observe the following safety advisories.
- Protect the battery against mechanical damage of any kind, e.g. powerful impacts or strong pressure. Do not try to open, repair, or otherwise manipulate the battery under any circumstances.
- Do not short out the battery. With the battery removed from the appliance, the battery contacts must not be connected by metal objects (e.g. hairpin, scissors or metal tail comb) under any circumstances.
- Do not expose the battery to extreme temperatures (below 0 °C or above + 40 °C). For instance, avoid proximity to heating devices or direct sunlight.

- Charge the battery only with the battery installed in the appliance, using the original accessories (charging unit or power cord) and observing the correct contact polarity.
- Charge the battery only in dry rooms and never in the vicinity of acids or easily flammable materials.
- Do not throw the battery into fire.
- Only use the original battery for this appliance.
- The battery must not be touched by water.
- Do not try to extinguish a burning lithium ion battery with water under any circumstances. Use sand or some other smothering substance for extinguishing the fire.
- In the event that electrolytic solution escapes from battery, avoid contact with skin, eyes and mucosa. Immediately rinse the affected areas with plenty of clear water and see a physician.

NOTE

If you are not going to use the appliance for any extended period, store the appliance or its battery, respectively, charged to 40-60 % of its capacity in a cool and frost-protected place (approx. 15 °C).

This measure serves to prevent premature aging of the battery.

2.7 Cleaning, care and storage

DANGER

The trimmer must be switched off and unplugged if:

- you want to clean or oil the appliance.
- you want to change the rechargeable battery.

Never attempt to remove dust or foreign bodies from inside the appliance with pointed objects (e.g. with the tail of a comb).

WARNING

To avoid excessive wear on blade set and motor, clean the blade set regularly and apply oil to the appropriate points (see "Cleaning and care" on page 17).

- Do not use aggressive or abrasive cleaning products to clean the appliance.
- Do not wind the power cord round the trimmer. This could break the cable.

- When the rechargeable battery has been removed, do not connect the trimmer to the power cord **11** or place it in the charging unit **8** that is connected to the mains. Either action can damage the electronic system.

3. Preparation


3.1 Charging

- Before connecting the trimmer to a power outlet, verify that the voltage stated on the rating plate is compatible with the power supply.

The appliance as supplied is not yet ready for operation with the rechargeable battery. However, you can use the trimmer immediately on mains power (see "Mains operation" on page 16).

The appliance should be charged for 2 hours before being used for the first time without the cable.

It is not possible to "overcharge" the appliance.

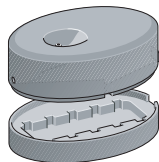- Before charging the trimmer for the first time, remove charging unit **8** from comb attachment holder **9**.
- Check that the mains voltage agrees with the voltage specified on the rating plate (on mains adapter **10**).


- Plug the power cord **11** into the charging unit and mains adapter **10** into a wall socket.
- Place the charging unit on a flat surface.
- Place the appliance in the charging unit.

The charging indicator **6** shows the charging status. It has 5 segments, each representing about 20 % of the capacity of the rechargeable battery.

-  Battery is low.
-  Battery is up to 20 % charged.
-  Battery is up to 40 % charged.
-  Battery is up to 60 % charged.
-  Battery is up to 80 % charged.
-  Battery is fully charged.

3.2 Storing the power cord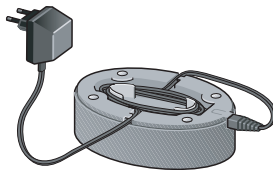

If you do not need the entire length of the power cord, you can stow the surplus in the compartment in the base of charging unit **8**.

3.3 Storing the comb attachments

You can keep the comb attachments **14** stored tidily in the comb attachment holder **9**.

4. Operation

4.1 Function lock

When it leaves the factory, the trimmer is protected against being switched on accidentally. You can activate this function lock yourself, e.g. for transporting the trimmer.

Deactivating function lock:

- Place the trimmer in the connected charging unit **8**. The charging indicator **6** lights up. The trimmer is ready for use.


Mains operation of the trimmer is possible immediately.

Activating function lock:

- Press the On/Off switch **3** for about 4 seconds. Using the On/Off switch now has no effect.

4.2 Mains operation

If necessary, e.g. if the rechargeable battery has little or no charge, you can run the trimmer on mains power.

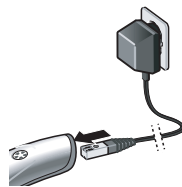- Insert the mains adapter **10** in a power socket.
- Unplug the plug of the power cord **11** from the charging unit **8** and plug it into the socket on the trimmer.

4.3 Switching On and Off

- To switch the trimmer on, press the On/Off switch **3**.
- To switch the trimmer off, press the On/Off switch **3** again.

4.4 Setting the cutting length

The cutting length is adjustable in three stages:


I: 0.5 mm

II: 1.5 mm

III: 2.5 mm

- Slide lever 5 to the desired position.

4.5 Comb attachments

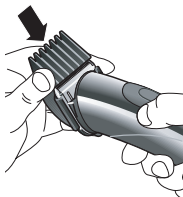The comb attachments 14 with the lengths 4, 8, 12, 16 and 20 mm can be pushed onto the blade set 4.

Please note: The cutting lengths of the comb attachment and the cutting length set on the trimmer must be added together.

Example: Comb attachment 20 mm and lever 5 in position III (2.5 mm) results in a cutting length of 22.5 mm.

- Switch off the trimmer and push the desired comb attachment onto the blade set 4.

You can now start trimming hair with the trimmer in the usual way.

5. Cleaning and care

To ensure proper functioning of the appliance, the blade set must be cleaned regularly and oiled in the appropriate places. Failure to clean the blade set regularly results in excessive wear on blade set and motor


Cleaning the blade set

Damage to the blade set caused by its disassembling in its individual parts!

- Never disassemble the blade set. Once disassembled it cannot be reassembled.

If used regularly, the blade set **4** should be thoroughly cleaned once a day.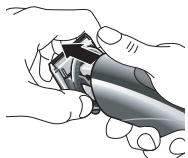

- Switch the appliance off.
- Push the entire blade set off the front of the trimmer.
- Use the cleaning brush **13** to remove any hairs.

- Slightly push apart the upper and lower parts of the blade set (see illustration) and use the cleaning brush to remove any hairs between them.

Oiling the blade set

The blade set **4** must be oiled regularly – preferably daily.


- Remove the blade set.
- Lubricate the areas where there is contact between the upper and lower parts of the blade set (see illustration).

- Run the trimmer for about 20 seconds to ensure that the oil is properly distributed. – You can carry on working if you want to.

Replacing the blade set

The blade set is subject to wear. Even if you look after the appliance properly, the cutting performance will deteriorate. Replace the blade set when it is worn. The blade set can be ordered separately as a spare part from Aesculap Customer Service (see addresses at the end of these instructions).

Attaching the blade set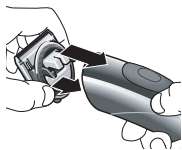

- Switch the appliance off.
- First position the lower lugs in the recesses in the trimmer, then press the blade set home while swivelling it upwards. Do not use force. You must hear the blade set click firmly into place.

Cleaning the appliance

- Clean your appliance occasionally with a damp cloth, then wipe it dry.

Cleaning the charging contacts

If the charging contacts **2** on the appliance or the charging contacts **7** on the charging unit are dirty, charging may not be possible or may be interrupted.

- Clean the contacts using a cloth or wad of cotton wool soaked in alcohol or spirit.

6. Changing the rechargeable battery

If the capacity of the rechargeable battery starts to fall off, you can order a new one from Aesculap Customer Service and insert it instead of the old one.


Use only the original Aesculap rechargeable battery which you can order from Aesculap Customer Service.

When disposing of the old battery, be sure to observe the disposal regulations (see "Disposal" on page 23).


WARNING

The power cord 11 must never be plugged into the trimmer when you are changing the rechargeable battery.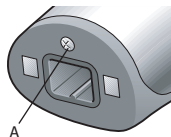


- Unscrew the screw **A** next to the socket **1** on the bottom of the trimmer.

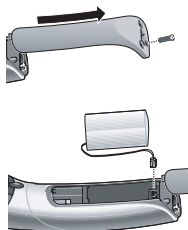- Slide the cover off the trimmer.
- Remove the old rechargeable battery from the trimmer and carefully detach the mini plug by pulling upwards.

When there is no rechargeable battery in the trimmer, do not connect the trimmer to the power cord 11 or place it in the charging unit 8 that is connected to the mains. Either action can damage the electronic system.

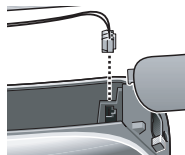- Carefully insert the mini plug of the new rechargeable battery into the trimmer so that it fits into the appropriate socket.
- Insert the new rechargeable battery, slide the cover back onto the trimmer, and screw up the securing screw.

7. What should I do ...

... the appliance will not switch on?

- Is the function lock activated? Place the trimmer in the connected charging unit to deactivate the function lock.
- Is the battery low? Charge up the trimmer in the charging unit or work with mains power (power cord plugged directly into the trimmer).
- Charging the trimmer at high ambient temperatures (>35 °C) may trigger the safety cutout, which temporarily stops the appliance working. Let the trimmer cool down for 5-10 minutes. Then the trimmer is ready for operation again.

... the blade set "sticks"?

- The blade set must be cleaned and oiled. Proceed as described on page 18.

... none of the charging indicator segments lights up?

- Clean the charging contacts.
- The appliance may not be positioned correctly in the charging unit. Position it correctly.
- The power cord may not be properly connected.
- The rechargeable battery is completely run down and is taking longer to charge.

... the appliance switches itself off?

- The battery is low. Charge up the trimmer in the charging unit or work with mains power (power cord plugged directly into the trimmer).

... the running time on one charge of the battery is much shorter than before?

- The blade set may need to be cleaned and oiled. Proceed as described on page 18.
- If this does not help, order a new rechargeable battery from Aesculap Customer Service and insert it instead of the old one (see "Changing the rechargeable battery" on page 19).

... the battery will no longer charge?

- Order a new rechargeable battery from Aesculap Customer Service and insert it instead of the old one (see "Changing the rechargeable battery" on page 19).

8. Technical data

Subject to technical change without notice.

Product	Akkurata/Vega
Mains adapter:	
Input voltage	100-240 V~, 50/60 Hz
Output voltage	6.5 V -, 900 mA
Battery	3.7 volt lithium-ion
Emission sound pressure level	LpA ≤70 dB(A)
Total vibration level	ah = 3.59 m/s ² k = 0.39 m/s ²
Cutting lengths (without comb attachment)	0.5; 1.5; 2.5 mm
Comb attachments	4, 8, 12, 16, 20 mm
Dimensions (L x W x H)	170 mm x 46 mm x 46 mm
Weight (trimmer)	<250 g

9. Disposal

This electrical appliance contains a lithium ion battery. The rechargeable battery must be removed before disposal (see "Changing the rechargeable battery" on page 19). The rechargeable battery must never be disposed of as household waste.

If the electrical appliance is no longer to be used, return it to the place where you bought it or hand it in to the special collection centre run by your local public waste disposal authority (recycling centre, materials recovery centre etc.).

The appliance must not be disposed of as household rubbish.

This also applies to electrical appliances that were last used in the non-private sector, e.g. in branches of trade or industry. In cases where public waste disposal authorities do not accept electrical appliances that have been used for non-private purposes, their waste advisory services can provide information on environmentally sound disposal.

10. United Kingdom

Kerbl UK Ltd.
8 Lands End Way
Oakham, Rutland
LE15 6RF UK
www.kerbl.co.uk
Phone: 0044 01572 722558

11. Two-year guarantee

Dear Customer,

Thank you for choosing a product from our company.

For decades, the name Aesculap has stood for proven quality and first-class service. We are known to provide innovative, high-performance products for our customers. Aesculap manufactures high-quality devices, using high-quality materials and careful manufacturing processes. We would like to point out to you that we are not liable for defects in our products, if these are caused by improper use, normal wear or are caused by the product being improperly used or if inappropriate or not properly functioning third-party products are used.

Defects that do not affect or only negligibly affect the value or function of the product are also excluded from our warranty.

If we are liable under our warranty, we reserve the right to repair or replace the product.

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Albert Kerbl GmbH