



PROFESSIONAL

**KM Cordless
Instructions**

Please read all instructions carefully to familiarise yourself with your new Wahl KM Cordless before using.

Save these instructions for further reference.

For any further assistance or information call Customer Services on

T: 01227 740066

or visit

W: www.wahl.co.uk

E: customer.services@wahl.co.uk

CONTENTS

GENERAL USER INFORMATION	Page 3
IMPORTANT SAFEGUARDS	Page 4
DESCRIPTION OF PARTS	Page 7
OPERATING INSTRUCTIONS	Page 8
CLEANING AND MAINTENANCE	Page 11
TROUBLESHOOTING	Page 13
BATTERY REPLACEMENT, REMOVAL & DISPOSAL	Page 14
WIRING	Page 15
GUARANTEE	Page 15

GENERAL USER INFORMATION

Information about using the operation manual

- ▶ Before using the appliance for the first time, you must read and understand the operation manual completely.
- ▶ Consider the operation manual as part of the product and store it in a safe and accessible place.
- ▶ Include this operation manual if passing the appliance on to a third party.
- ▶ In the case of conflicts with national safety specifications or instructions, the national requirements must be followed.

Explanation of symbols and notes



DANGER

Danger of electric shock which may result in serious physical injury or death.



DANGER

Danger of explosion which may result in serious physical injury or death.



WARNING

Warning of possible physical injury or a health risk.



CAUTION

Information about danger of material damage.



Note with useful information and tips.

- ▶ Action to be taken by owner.
1. Carry out these actions in the sequence described.
- List

Please retain this leaflet for future reference.

READ ALL INSTRUCTIONS BEFORE USING.

IMPORTANT SAFEGUARDS

When using this electrical appliance, especially when children are present, basic safety precautions should always be observed including the following.

Requirements

- ▶ Read the instruction manual in full to familiarise yourself with the product before the first use.
- ▶ This appliance can be used by children from aged 14 years and above and persons with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved. Children shall not play with the appliance. Cleaning and user maintenance shall not be made by children without supervision.
- ▶ Use this appliance only for its intended purpose.

HAZARDS

Danger! Electric shock due to damage to appliance

- ▶ Do not operate the appliance with a broken cord, plug or charger, or if the appliance malfunctions, or is dropped or damaged in anyway.
- ▶ Any servicing or repairs must be performed by Wahl (UK) Ltd or a suitably qualified professional.
- ▶ The use of attachments that are not supplied by the manufacturer may cause fire, electric shock or injury and damage to the appliance.



Danger! Electric shock due to penetration of liquid

- ▶ To protect against risk of electrical shock, never immerse the unit or let it come into contact with water or any other liquids. Do not use the unit with wet hands. If the appliance should become wet, damp or fall in water, remove plug from mains socket immediately. Do not put your hands in the water. Do not take into bathroom.



Danger of explosion

- ▶ Never use the appliance where aerosol (spray) products are being used or where oxygen is released.
- ▶ Do not expose the appliance to temperatures below 0°C and above +40°C for any length of time. Avoid direct exposure to sunlight.



Warning! Injuries due to incorrect handling

- ▶ Cutting blades may become hot after prolonged use.
- ▶ Switch off the appliance and unplug from outlet when not in use, before putting on or taking off attachments and also before cleaning the appliance.
- ▶ Never use your appliance without paying attention! Except when charging always unplug this appliance from the electricity outlet immediately after using.
- ▶ These appliances are intended for animal hair trimming and clipping only.



Caution! Danger of product damage

- ▶ Do not wrap the cord around the appliance.
- ▶ Do not use outdoors or in damp areas.
- ▶ Never carry the unit by its power cord.
- ▶ Do not drop or insert any object into any opening.



OPERATING INSTRUCTIONS

DIRECTIONS:


- ▶ Charge the clipper battery fully before using for the first time.
- ▶ This clipper is equipped with two different operating speeds. Turn the clipper on by sliding the on/off switch upward. Position "1" runs at 3,000 strokes per minute (SPM). Position "2" runs at 3,700 strokes per minute (SPM). After use, switch the clipper off by sliding the on/off switch back down to Position "0".


CHARGING THE BATTERY


- During cordless operation, the LED light will light up solid green. When the green LED light begins to flash, this indicates battery charge is low and the unit will need to be recharged.
- When the battery has fully run down, the clipper will switch itself off automatically.
- To charge, connect the cord to the charging socket on the clipper and then plug the plug-in transformer into an electrical socket.
- When the battery begins to charge, the LED light will blink green to indicate the unit is charging. When the LED light has stopped blinking and remains a constant, unblinking green, the clipper is fully charged and ready for use.
- Full charge time will take approximately 60 minutes. A full charge allows for approximately 120 minutes of clipper run time.
- In the case of low battery, the clipper can be used as a corded unit. However the battery will not begin to charge until the clipper is turned off.
- This clipper includes a "Quick Charge" feature; For example, 10 minutes of charge time will provide 15 minutes of run time

GETTING THE MOST FROM YOUR BATTERY

 Only plug in your clipper when charging is needed.

 If your battery is dead, you can plug in your clipper and keep using it.

 It is fine to partially charge your battery because it does not require a full charge.

 It is best NOT to leave it plugged in all the time! If your clipper battery is dead, charge it as soon as possible.

Store and charge your clipper at cool, dry temperatures away from heat sources or direct sunlight. If you are storing your appliance, get it out every 6 months and give it partial recharge.

Extend the life of your clipper battery by keeping the blades clean and oiled.




BATTERY TROUBLESHOOTING

- **Flashing Red LED Light:** The over-current protection circuit has been activated, possibly due to using a dirty or scored blade. The unit will automatically shut down, but can be reset by turning the switch to the off position and then turning it back on. If the problem is not resolved, the unit may stop again right away.
- **Solid Red LED Light:** The battery pack temperature is too high. This could happen if the unit is used under very heavy clipping loads in extreme temperature conditions or while charging the unit in an extreme temperature environment. The unit will be temporarily disabled until the battery temperature returns to a normal level. Do not attempt to recharge the unit until the solid red LED light has turned off. If a problem persists, please contact Wahl Customer Service.

i **NOTE:** We perform quality assurance on all of our products and small bits of synthetic hair may appear on the blade or clipper as a result of this process. Rest assured that your clipper and blade are brand new!

CHANGING THE BLADE SET

1. Switch off the clipper using the on/off switch.
2. Locate the blade release button on the backside of the clipper (Fig 1a). Press in the blade release button and gently pull the blade away from the clipper (Fig 1b). Slide the blade off the blade latch (Fig 1c).
3. To place a new blade on the clipper, ensure that the blade latch is in the correct, upright position, not lying flat against the clipper.
4. If the blade latch is not upright, you will need a flathead screwdriver to pry it up. Use the flathead screwdriver to pry open the blade latch from the side. Do not attempt to pry the blade latch open from the top as there is a chance the drive tip could be damaged.
5. Insert the blade onto the blade latch by sliding the blade retainer over the blade latch (Fig 3a). Before snapping the blade into place, turn the clipper back on. It is important the clipper be running as this ensures the blade will seat properly. Use the palm of your hand to guide the blade to the clipper (Fig 3b). There will be an audible "click" when the blade is set in place.

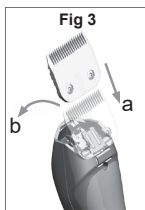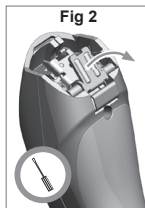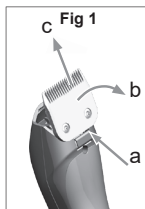

CUTTING WITH THE ATTACHMENT COMB



The appliance can also be operated with attachment combs. Attachment combs can be used to extend the cutting length range.

For best results we recommend using a #30 or #10 blade with the comb attachments.

Accessories are available from your local dealer.

ATTACHING/REMOVING THE ATTACHMENT COMB

1. Fit the attachment comb onto the bottom edge of the blade set (Fig 4a) and push it over the cutting edge of the bottom blade until it locks (Fig 4b).
2. Push the attachment comb towards the cutting edge of the bottom blade (Fig. 5) and remove it from the blade set.

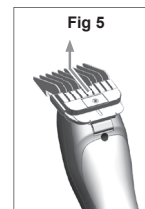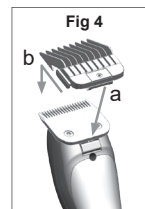


CLEANING AND MAINTENANCE

Do not immerse the clipper in water!

1. After each use, use the cleaning brush to remove loose hair from the clipper housing and the blade set.
2. The blade set should be oiled frequently in order to maintain its good cutting performance and thus ensuring a long service life. Only use Wahl Clipper Blade Oil. Wahl Clipper blade oil has been specifically formulated for Wahl Clipper blades. Using any other lubricant may cause damage to your blades and clipper and would void your clipper's warranty.

3. Properly oil your blade using the 5 Point Oiling Method (Figure 6). Place 3 drops of oil along the top of the blade teeth at the left corner, right corner, and centre of the blade. Turn the clipper to its side and place a drop of oil at the heel of the top blade. Repeat on the other side. Let the clipper run for 30 seconds to allow the oil to work its way into the blade set.

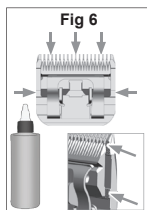

4. Wipe away any excess oil. Only wipe the clipper with a soft cloth, which may be dampened slightly. Do not use solvents or abrasives!
5. The blade set may be cleaned with Wahl Clipper hygienic spray. Do not use any other cleaning agent. Wahl Clipper hygienic spray has been specifically formulated for Wahl Clipper blades. Using any other cleaning agent may cause damage to your blades and clipper and would void your clipper's warranty.



Caution! Danger of product damage

Insufficient maintenance of the blade set can reduce the operating time considerably. Care should be used to keep the cord untwisted and damaged.



Caution! Danger of product damage

When storing the clipper, coil the cord. Wrapping the cord around the clipper for storage can cause strain to the cord, which over time, can lead to the loosening of the wires and/or electrical shortages.

TROUBLESHOOTING

PROBLEM	POSSIBLE CAUSE	SOLUTION / HINT
Blade set is cutting badly or pulling hair out.	Blade set is dirty, worn out or blade is blunt.	Clean and oil the blade set. If this does not resolve the problem, replace the blade set.
Skin injury	Too much pressure on the skin.	Reduce pressure when cutting close to skin.
Skin injury	Damaged blade set.	Check whether the blade edges, or teeth are damaged and replace the blade set if necessary. NEVER use a damaged blade as this will cause injury.
Appliance is not working	Defective power supply.	Make sure that contact between the mains and the socket is faultless. Check the mains cable for possible damage.



In the event that this information does not help you resolve your problem, please contact our Customer Service team. Never attempt to repair the appliance yourself!

WASTE ELECTRICAL & ELECTRONIC EQUIPMENT DIRECTIVE 2012/19/EU (WEEE)



Within the European Community, national regulations are specified for the disposal of electrical appliances, based on EU Waste Electrical and Electronic Equipment Directive 2012/19/EU (WEEE). In accordance with this, the appliance can no longer be disposed of with the local or domestic waste. The appliance will be accepted free of charge by local collection points or recycling centres.

LITHIUM BATTERY REPLACEMENT, REMOVAL AND DISPOSAL

Lithium-Ion Battery Replacement

For battery replacement the entire, intact clipper or trimmer should be sent to: Wahl (UK) Ltd Customer Services (the address can be found on the reverse of this booklet) or taken to a suitably qualified professional. The battery should only be replaced by the original manufacturers spare, for safety reasons a substitute should not be used.

Do not attempt to remove the battery. Lithium-Ion batteries can explode, catch fire, and/or cause burns if disassembled, damaged or exposed to water or high temperatures.

Lithium Battery Disposal

Do not dispose of this product or the Lithium battery in the normal household waste.

The product should be returned to Wahl Customer Services where we offer a take back service or taken to a local collection facility where they will be dealt with safely and sent for the appropriate recycling.

DISPOSAL



Handle with care. Environmental damage in the case of incorrect disposal.

Correct disposal will ensure environmental protection and prevent any potentially harmful impacts on people and the environment.

Adhere to the relevant legal requirements when disposing of the appliance.
Information on the disposal of electrical and electronic appliances in the European Community.

WIRING



WARNING:

If the transformer is damaged it must be replaced in order to avoid a hazard.

This product must only be used with the transformer supplied. Should a replacement be required contact Wahl Customer Service.

GUARANTEE

This product is guaranteed against defects in material and workmanship for a period of 1 year from the date of original purchase or receipt as a gift.

Should this product become defective during the guarantee period, return it to the store of purchase together with your proof of purchase for repair or replacement

Alternatively within the guarantee period you can return the product to Wahl (UK) Ltd who will repair any such defect or elect to replace the product or any part of it, without charge, provided that there is proof of purchase. Should a replacement be offered this will not extend the original guarantee period.

The guarantee becomes invalid in the case of misuse, alteration or repair by unauthorised persons.

This guarantee does not include blades, cables, and cutters etc, which are consumable parts.

This guarantee in no way affects your rights under statutory law in the United Kingdom.

WAHL (UK) LTD
STERLING HOUSE
CLIPPER CLOSE
RAMSGATE, KENT
CT12 5GG

CUSTOMER SERVICES T: 01227 740066
www.wahl.co.uk

Form No. ZB816
Rev.2 08-20