

Quick set up for the X Series



Please contact the Technical Team if you have any questions.
Email: tech@vet-direct.com

| | |
|---|---|
| 1 | Patient Setup – Will appear when first turned on (New Patient or Current) – or Menu > Patient Setup – New Patient if wanting to input Patient name, operating vet, weight, age etc. Quick Admit if just wanting to choose species and blood pressure cuff size. |
| 2 | Blood pressure cuff settings – Small (two smallest sizes of cuff) Middle (other two sizes of cuff between largest and two smallest) Big (largest cuff). |
| 3 | Blood pressure Auto or Manual? – Press on the NIBP module on the main screen > Measure mode to Manual, you will have to press the NIBP button to take a reading. Auto > Interval, you can then select how often in minutes a reading will be taken. Continuous readings can be done but this drains the battery and isn't suitable long term for your patients. |
| 4 | Profiles set up: Click on a module > Alarm Setup > Set alarms high and low accordingly to patient species/size > once happy with alarms press Menu > Maintenance > User Maintain > Password (you will have set this when first using the unit) > Profile > Save > Name the profile (ie. Cat, Small dog, Large dog, Exotic etc). |
| 5 | Choosing a Profile – Menu > Profile > List of saved Profiles will be here for you to select from. When the unit has been turned off the last used Profile will be open, this will not change when you admit New Patient. You have to select from the list of saved Profiles. |
| 6 | Screens – Shortcut bar along the bottom has four different screen setups, Standard, TrendScreen (standard screen with a table which shows your readings for the last half an hour) Large Font (Small wave with large font on the readings) Vital (ECG, NIBP, SPO2 only). |

List of Suggested Consumables



VEP009 & VEP009B



11.57.4271157 – 11.57.471161



PFAC001

- VEP009 – SP02 large clip
- VEP009B – SP02 small clip

- 11.57.471157 – Edan Neonatal #1 Blood Pressure Cuff (3–6cm)
- 11.57.471158 – Edan Neonatal #2 Blood Pressure Cuff (4–8cm)
- 11.57.471159 – Edan Neonatal #3 Blood Pressure Cuff (6–11cm)
- 11.57.471160 – Edan Neonatal #4 Blood Pressure Cuff (7–13cm)
- 11.57.471161 – Edan Neonatal #5 Blood Pressure Cuff (8–15cm)



ABS001



01.57.471275



02.01.210520

- 01.57.471275 – Edan iM60/iM70/X Series Disposable CO2 Sampling line with male luer lock (DM-1100)
- 02.01.210520 – Edan iM60/iM70/X Series Dewatering Cup (Water Trap)

- PFAC001 – ECG Atraumatic crocodile clips
- ABS001 – ECG pads

Find all Edan equipment and consumables online on our browse by brand page

